

ZAMST HA-1 INSTRUCTIONS FOR USE

HA-1

Please read through the following instructions completely before using the product.

*This product is for arch support; however, product fit varies by individual.

*This product does not prevent or heal foot injuries.

How to apply:

The socks are designed specially for the right and left feet. Make sure to wear the correct sock on each foot.

Precautions:

- To prevent adverse symptoms, do not use the socks in the following cases:
 - If your skin is sensitive or prone to allergies.
 - If you have abnormal conditions such as cuts, fractures, numbness, swelling or rash in the area where product is used.
- Follow the instructions below and use the product correctly to avoid poor blood circulation or injuries.
 - Use the correct size. (Refer to the size chart on the box.)
 - Do not use the product while sleeping.
 - Do not use the product for extended periods.
- If you experience the following adverse effects, discontinue use immediately and consult with a physician.
 - If you experience discomfort while wearing the product.
 - If you experience cuts, fractures, pain, numbness, swelling or a rash in the area where this product is used.

Caring for the product:

- Use a laundry bag when you clean the product in a washing machine.
- The product's dye may bleed. Do not wash with whites or bright colors.
- Do not place in dryer.

Composition:

Cotton, acrylic, polyurethane, nylon and polyester